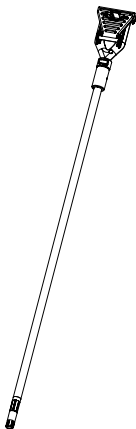
# **Baxter**

## **Gasless Tip APPLICATOR**

Instructions for Use  
Mode d'emploi  
Gebrauchsanweisung  
Instrucciones de uso  
Istruzioni per l'uso  
Gebruiksaanwijzing  
Bruksanvisning  
Instruções de utilização  
Инструкции за употреба  
Návod k použití  
Brugsanvisning  
Οδηγίες χρήσης  
Käyttöohjeet  
Upute za uporabu  
Instrukcja użytkowania  
Kullanma Talimatı  
Bruksanvisning

**DIST**

Baxter Healthcare Co.  
Deerfield, IL 60015, USA  
1-888-229-0001

**Manufactured in Mexico for:**

Micromedics, Inc.  
1270 Eagan Industrial Road  
St. Paul MN 55121-1385, USA  
651-452-1977

**CE 0297**

**EC REP**

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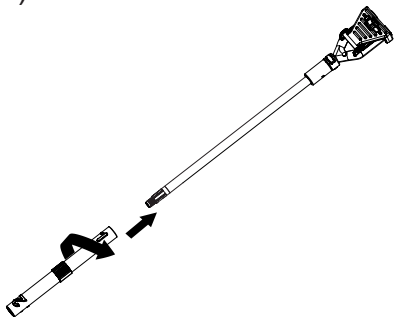
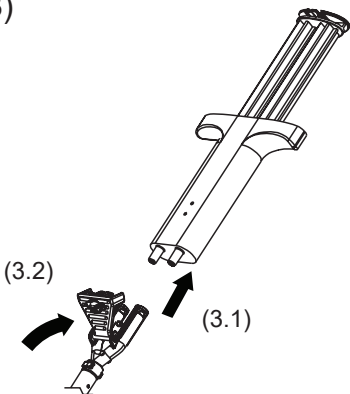
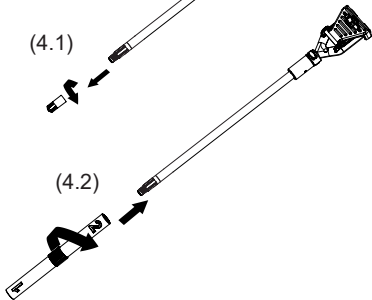
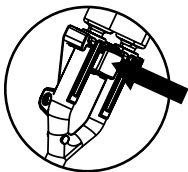
APPLICATOR

EN

**Before using, read the following information:**

## **DESCRIPTION:**

The Gasless Tip is a sterile, single use dual lumen applicator. Each pouch contains 1 applicator and 1 tip alignment tool with 2 pre-loaded spray tips.

## **INTENDED USE:**

Surgical Applicator

## **INDICATIONS FOR USE:**

The Gasless Tip is indicated for the application of Tisseel Fibrin Sealant.

## **CONTRAINDICATIONS:**

None

## **WARNING/PRECAUTIONS:**

- Do not spray directly into circulatory pathways. Not for intravascular application.
- Application of the liquids must be performed under visual control.
- Single use only. Do not attempt to re-sterilize or reuse device.
- Device may degrade if reprocessed. No effective cleaning process has been developed to allow reuse of device. Reuse of device may lead to injury or illness of a patient.

## **INSTRUCTIONS FOR USE:**

1. Remove the applicator from the sterile package using sterile technique.
2. Using the Tip Alignment tool with pre-loaded spray tip, thread the spray tip onto the end of the applicator in a clockwise motion as shown in Figure 1. Ensure the tip is secured firmly to prevent accidental tip detachment, but do not overtighten.  
  
(Note: If a drip application is desired, do not attach spray tip.)
3. Open the snap lock by pressing the release button as shown in Figure 2.

4. Push the male syringe set Luers firmly into the female Luer cones on the applicator as shown in Figure 3.1.
5. Push down on the snap lock to securely fasten the applicator to the syringes Luers as shown in Figure 3.2.
6. The applicator is ready to use.
7. For spray application, apply the liquid components by depressing the syringe plunger with firm, rapid pressure until the desired spray pattern is achieved. For drip application, apply the liquid components using slow, steady pressure.
8. If the spray tip becomes occluded during use, pause product application. Unscrew the occluded tip by applying a counterclockwise motion as shown in Figure 4.1, wipe off any existing material within the distal end of the applicator. Use the tip alignment tool to replace with another tip following the instructions above as shown as Figure 4.2. Ensure the tip is secured firmly to prevent accidental tip detachment, but do not overtighten. Applicator is ready for use.
9. To release the applicator from the syringe Luers, press the release button on the snap lock as shown in Figure 5 and pull the applicator off the syringe set.
10. Dispose of used devices as biohazard waste, following local regulations.

*Baxter, Tisseel are trademarks of Baxter International Inc., or its subsidiaries.*

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*Baxter och Tisseel är varumärken som tillhör Baxter International Inc. eller dess dotterbolag.*